



## PB560 Ventilator Client Version




To Access Parameter Menu if locked:

Press and hold up and down arrow keys for 6 seconds

To Make Parameter Changes:

1. Highlight the parameter you want to change with the black box on the left of the parameter using the up/down arrow keys
2. Press the check mark-a +/-sign appears beside the parameter to be changed and the parameter flashes
3. Using the up/down arrow, adjust to your desired setting
4. Press the check mark key to accept the change (must be done within 7 seconds or the parameter reverts to its previous setting)

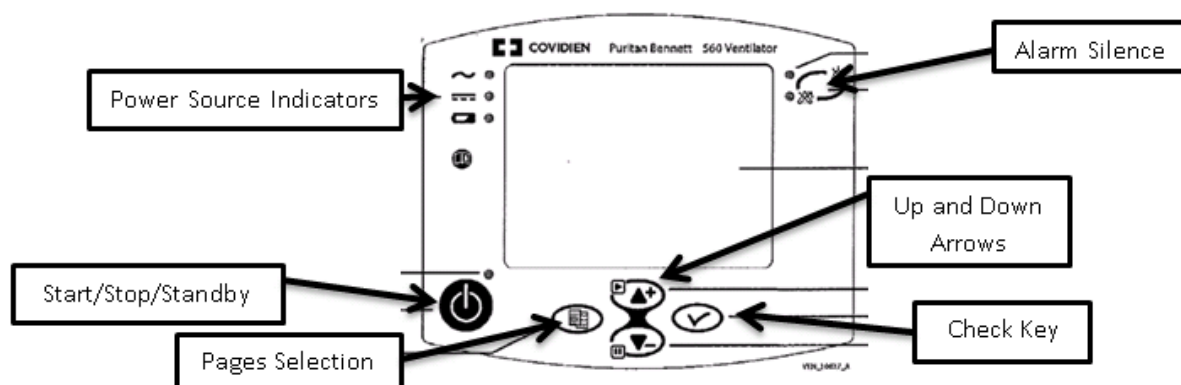



Filters to use: ID # 308; white side faces out, all sides must be fully inserted into the air intake area; check weekly, change as needed (disposable)


Power Supply: A/C, D/C, Internal Battery

**Internal battery** (length of time depends on ventilator settings-4-6hrs generally)

- displayed as a battery symbol in the top middle of the display screen initially in % of battery life remaining
- LED is illuminated beside the battery symbol on the left hand side
- After a few minutes of ventilation on set parameters, the % changes to an amount of time ie. 6h30
- **NEVER LET IT GO BELOW 20% BATTERY LIFE FOR SAFETY OF THE CLIENT**
- Continuous alarm and "Empty Battery" displayed at 5%
- "End of Battery" displayed at 0-5%
- Device must be immediately plugged in to external power source
- Battery takes 8hrs to recharge

Humidifiers: active humidifier with bedside set up; Heat Moisture Exchanger is used with the wheelchair set up if **no** Passy Muir Valve/Speaking Valve is in use

Alarms: The only two alarms PROP routinely sets in ACV are high and low pressure

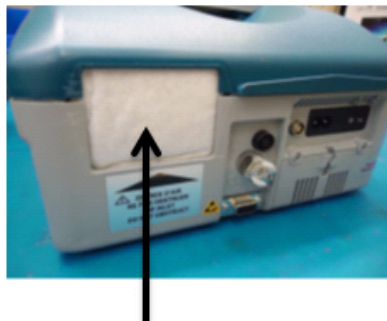

Proper placement of air inlet filter


To **start Ventilation**, press the Start/Stop key on the bottom left corner of the unit. The blue LED light above it will turn off.

To **stop ventilation**, press and hold the Start/Stop key for 3 seconds, release key when prompted, press the on/off key again to confirm the intentional stop of ventilation. Blue LED light will turn on to indicate the vent is in standby mode.



Parameter Page

Alarm Page

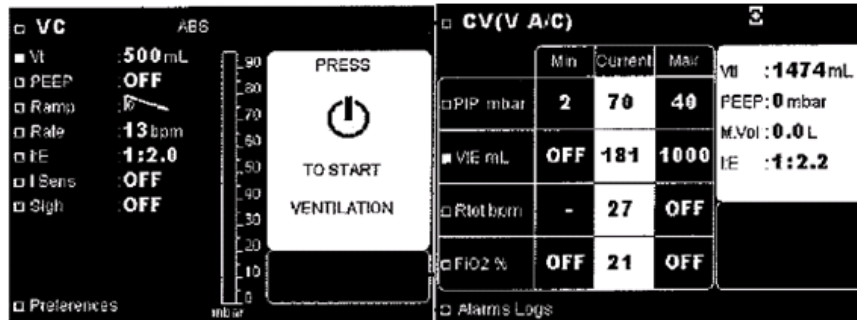

Min= Low Pressure Alarm  
Max=High Pressure Alarm

Use the Pages Selection button to the left of the up/down arrows to switch between these two screens

PROP is funded by the Province of British Columbia Ministry of Health through Vancouver Coastal Health.